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Feme Pad Instructions

Instructions for using Femé Pads® Suitable for repeated use by one individual:

1. Place a gel pad in the poly bag provided in a freezer for a minimum of 2 hours prior to use.
2. Remove the gel pad from the poly bag, insert it into a gauze sleeve and apply carefully to the wound.
3. After use, discard the gauze sleeve.
4. The gel pad must be cleaned in hot soapy water and dried thoroughly with a clean towel before replacing in the poly bag and returning to the freezer.

Safety Warning:

Keep out of reach of children.

Do not use sterilising fluid/tablets to clean the gel pad.

Should the gel pad become punctured or damaged in any way discard immediately.

Gauze Sleeves are also available in packs of 30.

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Manufactured Under Licence by Florri-Femé Pharmaceuticals Limited, Leeds LS1 6ER.
www.florri-feme.co.uk

Information about the Femé Pad[®] for Mothers

What is Femé Pad[®]?

The Femé Pad[®] is specially designed to give women instant pain relief in the vaginal and rectal region. The pain can be caused by vaginal irritations, injuries sustained during childbirth and inflamed haemorrhoids (piles).

When should I use Femé Pad[®]?

The Femé Pad[®] can be used during pregnancy, as well as after having a baby, to relieve vaginal irritations and soreness. A woman's perineum (the part of the body between the opening to the vagina and the back passage) will stretch and may tear during childbirth, sometimes it is necessary for a midwife or doctor to perform a cut (episiotomy). These two types of injury may involve stitches. Swelling and bruising occurs very quickly and a woman may have to cope with the pain and discomfort for several days. Usually a woman will be advised to take painkillers and bathe, an ice pack may be given to try and alleviate the pain and discomfort by numbing the area but some women have found them uncomfortable and difficult to wear.

Why Femé Pad[®]?

The specially designed Femé Pad[®] provides a more comfortable, easy to use cooling alternative. Its soft cushioning effect can also relieve pressure when sitting down, which will enable a new mother to sit more comfortably to feed her newborn baby.

Benefits of Femé Pad[®]

By applying the Femé Pad[®] directly to the affected area, it has the ability to stop the unpleasant throbbing pain that can be very distressing for a lot of women. The Femé Pad[®] has many benefits for a woman; it is simple to use both in the hospital and in her own home, it is noninvasive, drug free, discreet and comfortable.

A woman can be reassured that it is safe to use and will not have any adverse effects on herself or her baby. Two Femé Pads[®] are required, (one on, one in the freezer), for a woman to achieve maximum benefit.

Cleaning them could not be simpler; washing them with hot soapy water and drying them before being placed in the poly bag provided for re-cooling is all that is required.

This woman friendly product has been specifically designed with the help and support of midwives and women working together in partnership. It is a natural way to ease the pain and discomfort following childbirth, therefore, making early motherhood much more enjoyable.

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Fabriqué sous licence par Florri-Femé Pharmaceuticals Limited, Leeds, UK LS1 6ER.